


# 手术显微镜安装指南

operation microscope installation instructions

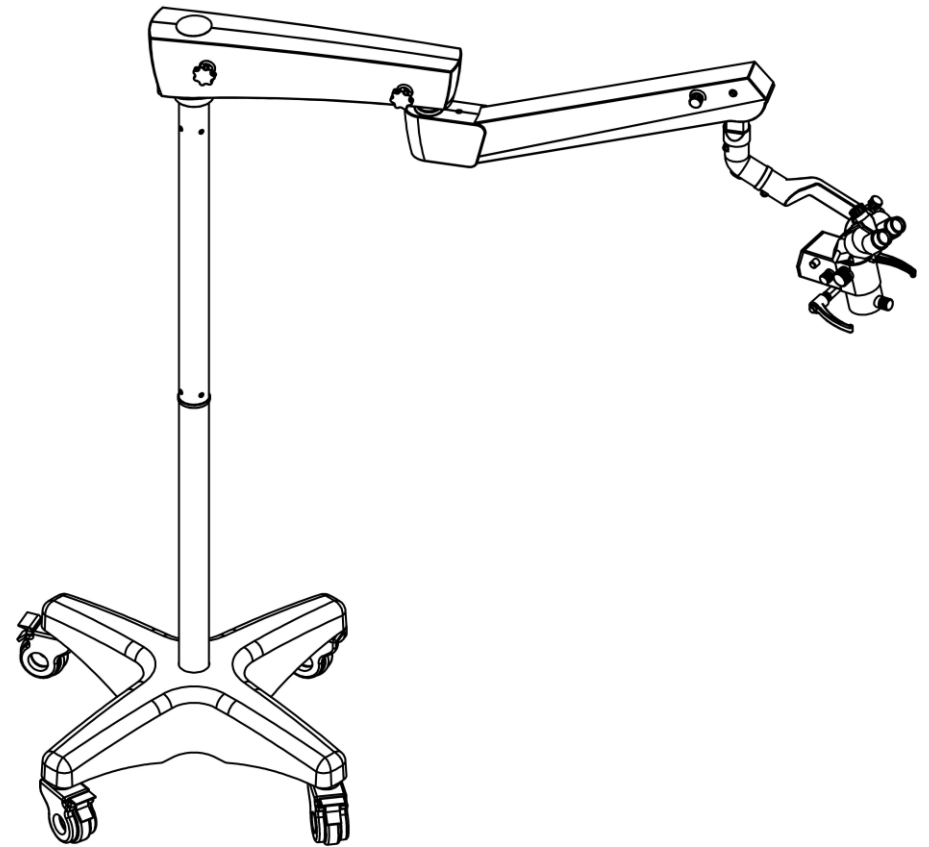




### 第五步:显微镜主体安装

**The fifth step:** Install-ling the main body of the operating microscope

## 警示 warning

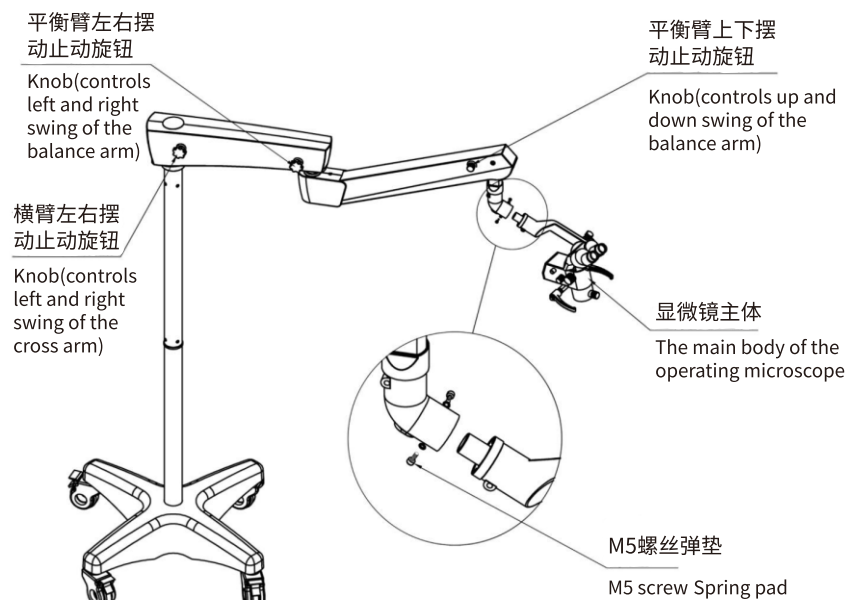

### 第六步:连接电源线即完成

**The Sixth step:** Connect the power cord to complete.

### 第七步:调试

**The Seventh step:** debugging

1.依次检查各锁紧手轮、转动件是否转动正常,并能有效锁紧。

1.In order to check whether the locking hand-wheels and rotating parts rotate normally and can be effectively locked.

2.将各电源开关处于关闭状态,接通电源,打开电源开关。

2.Turn each power switch in the off state, turn on the power, and turn on the power switch.

3.参照使用说明书检查各使用功能是否工作正常。

3.Refer to the instruction manual to check whether each function is working properly.

**说明:**若手术显微镜选配有照相机、显示器等配件,其安装说明请参照《相机、显示器安装说明》

**Note:** If the operating microscope is equipped with accessories such as cameras and displays, please refer to the "Installation Instructions for Cameras and Displays" for installation instructions.

1.安装、调试人员应接受相应的培训,安装前应充分阅读并理解设备的使用说明、安装说明及注意事项;  
1. Installation and commissioning personnel should receive corresponding training, and they should fully read and understand the instructions for use, installation instructions and precautions of the equipment before installation;

2.本安装说明书仅供参考,不能替代安装、调试的必要培训;  
2. This installation manual is for reference only and cannot replace the necessary training for installation and commissioning;

3.安装、调试、使用应在干燥、清洁的环境下进行,地面应平整,若地面不水平,其倾斜度应小于5°;  
3. Installation, debugging, and use should be carried out in a dry and clean environment. The ground should be level. If the ground is not level, its inclination should be less than 5°;


4.安装、调试过程中请注意保护好设备,避免设备的零件结构受到损伤;  
4.Please take care to protect the equipment during installation and commissioning to avoid damage to the parts and structures of the equipment;

5.安装相机和显示器时应带手套(已随机配2双),镜头镜片,显示屏尽量不要触碰,以免镜片和显示屏产生渍痕。需拆相机时,应关掉电源。  
5. When installing the camera and the monitor, you should wear gloves (2 pairs have been randomly equipped), and try not to touch the lens lens and the display screen to avoid stains on the lens and display screen. When you need to disassemble the camera, turn off the power.


6.安装中出现任何异常现象请及时与厂家售后服务人员取得联系;  
6. Please contact the manufacturer's after-sales service personnel in time for any abnormal phenomenon during installation;

手术显微镜安装步骤：  
operating microscope installation steps:


第一步：底座与脚轮安装  
First step: Base and caster installation

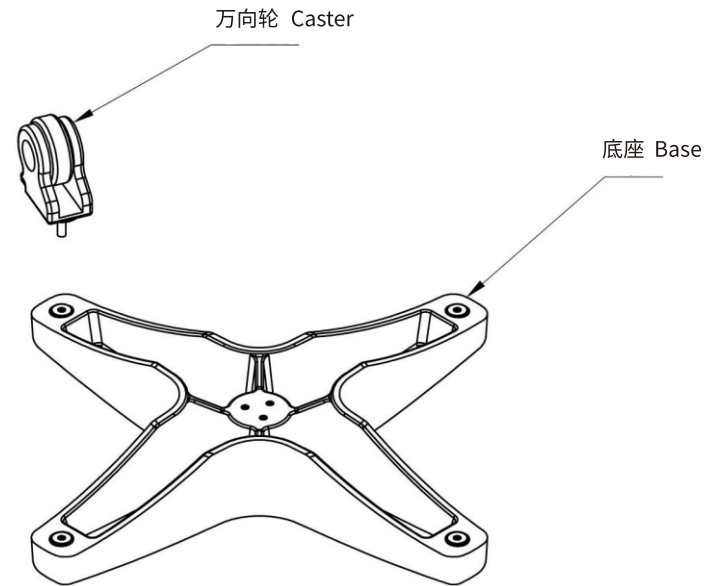

第二步：立柱II与底座安装  
Second step: Column II and base installation

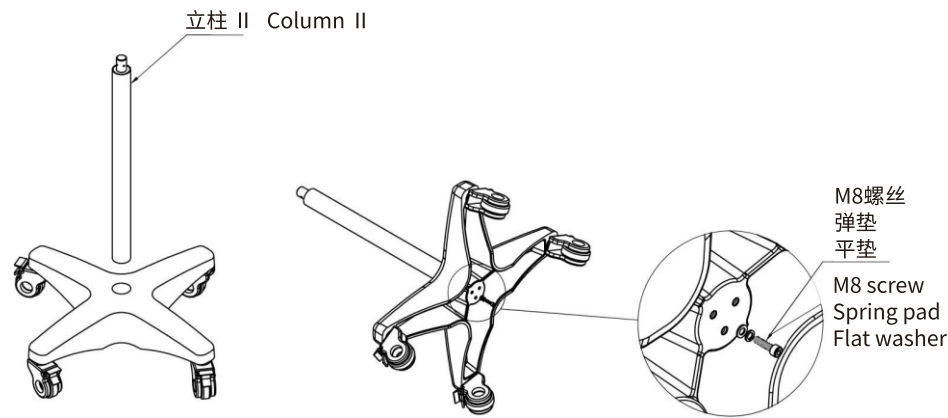

第三步：立柱I安装  
Third step: Column I and installation

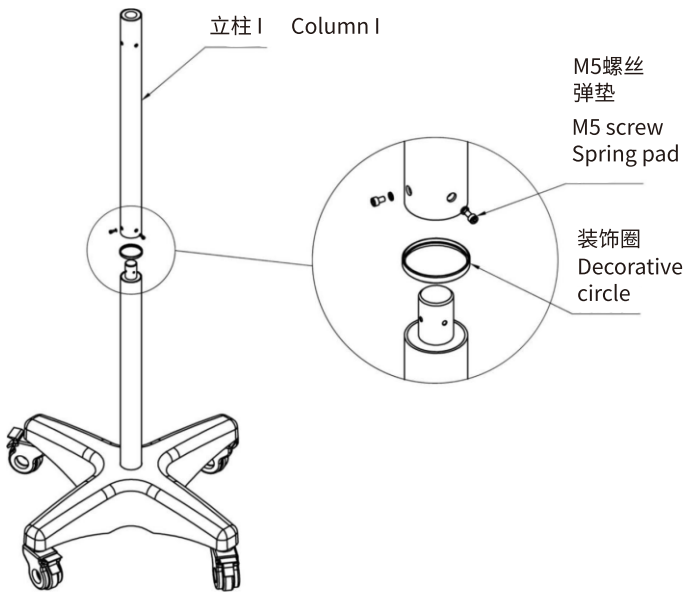

第四步：横臂、平衡臂安装  
The fourth: Cross arm/Balance arm installation

